



# Tetanos Quick Stick TQS

**Immunochemical test  
for the rapid detection  
of anti-tetanus antibodies in  
human serum,  
plasma or whole blood**



ZenTech

## ORDERING INFORMATION

Code : **S-GZ-20B**  
Package Size : 20 tests/kit

Code : **S-GZ-40B**  
Package Size : 40 tests/kit



## Indications

- Determination of the real immune status
- Identification of unprotected individuals
- Prevention of side reaction to revaccination
- Follow up of immunodeficient patients

## Features

- Ready to use (no predilution of the samples)
- **Fast result : 10 minutes**
- Easy interpretation of results
- Reliable answer
- Cost saving, no need for injection if the test is positive
- Suitable for physician's office, clinical labs or emergency rooms

## Performance characteristics

### Sensitivity

- Detection limit
  - 0,1 IU/ml on serum (WHO Standard)
  - 0,2 IU/ml on whole blood

### Specificity

- 100 % correlation with ELISA

## Storage and stability

- Storage in dry place between 4°C to 30°C (39°F and 86°F)
- Long shelf life : 18 months

## Kit contents

- 20 or 40 tests units individually packaged including disposable plastic pipettes
- 1 or 2 diluent vial(s) of 5 ml
- 1 instruction leaflet




Zentech s.a.  
Liège Science Park  
Avenue du Pré-Aily,10  
B-4031 Angleur • Belgium  
Tel : +32 (0)4 361 42 32  
Fax : +32 (0)4 367 00 63  
info@zentech.be  
<http://www.zentech.be>



ZenTech

**PEDIATRICS**  
**PEDIATRICS**

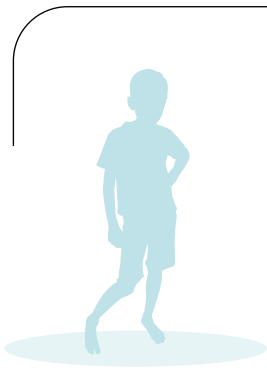


# Tetanos Quick Stick

## TQS


### Procedure




Blood is obtained by finger puncture with a safety lancet.

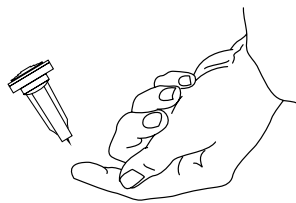


Fill the precalibrated plastic pipette with specimen  
⚠ correct blood volume (20 µl)

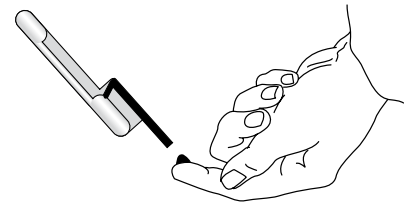


**1** Dispense the total volume of sample in the sample well

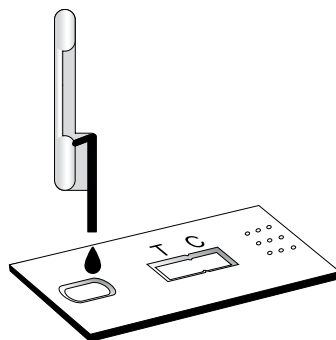

**2** Within 10 seconds, add 3 drops of diluent in the sample well

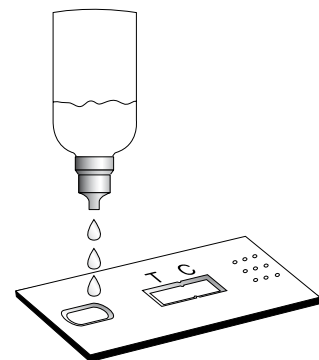


**3** Read results after 10 minutes

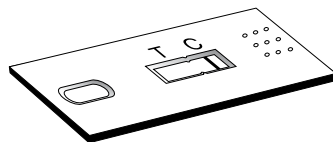

#### Negative test

*One coloured band appears in control window (C)*

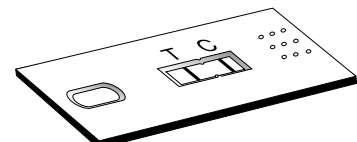

#### Positive test

*Two coloured bands appear in test (T) and control (C) window*

Zentech s.a.  
Liège Science Park  
Avenue du Pré-Aily, 10  
B-4031 Angleur • Belgium  
Tel : +32 (0)4 361 42 32  
Fax : +32 (0)4 367 00 63  
info@zentech.be  
<http://www.zentech.be>